



DIGITAL CONNECTION

MEDIT

Log into www.meditlink.com with the administrator account
Go to "Partners" on the left
Search for PNWproslab.com and click the "Request Partnership" button
You'll see that the lab has been added to the "My Partner List" with the status reflected as "Partnership Pending".

TRIOS (3 shape)

Option 1

Go to <https://us.3shapecommunicate.com>
Select "CONNECTIONS"
Request a connection with PNWproslab.com
Call us at (425) 698-1440 to let us know you have requested a connection
After your connection to PNWproslab is approved, select it when sending files

Option 2

Send an email to us at www.HYPERLINK "<http://pnwproslab.com/>"nwproslab.com, or call us to provide our team with your email address, and we can send you an invitation to connect.

ITERO

Option 1

Select "Find a Laboratory" on your iTero scanner
Connect with PNWproslab.com
After connecting to us as a Lab, select it on your scanner when sending files

Option 2

Call 800-577-8767
Select Option 1
Request that PNWproslab is added to your scanner. Identify our lab using our phone number (425) 698-1440
After PNWproslab has been added, restart your scanner
After connecting to us as a Lab, select it on your scanner when sending file

CEREC SIRONA

Log in to Cerec Connect at www.sirona-connect.com
Select "My Cerec Connect," then "Edit Account"
Under "My Account," select PNWproslab.com
Type PNWproslab in the field for "Company Name," then 98005 for Zip Code
Click "FIND"
Check the box that says PNWproslab
Click "ADD" to complete the registration
When sending a case, add "Send to PNWproslab" in the notes